



SafetyLancets



mylife™ SafetyLancets

Capillary blood sampling with reduced risk of accidental needle stick injuries

The innovative patented design makes capillary blood sampling safe and easy. The single release function and a sealed protection cap offer security and protect patients as well as healthcare professionals from cross-contamination or injury. mylife™ SafetyLancets are available in two different versions: mylife™ SafetyLancets (28G/1.5 mm) and mylife™ SafetyLancets Comfort (30G/1.2 mm).

Safety features

- The safety seal visually indicates whether the mylife™ SafetyLancet has been used before or not
- The protective cap maintains sterility and prevents contamination prior to use
- Inoperable after use thanks to single release function



mylife™ SafetyLancets
(28G/1.5 mm)


Comfort features

- Superior needle quality (tri-beveled needle tip) supports an optimum blood flow with maximum comfort
- No pre-loading or assembly prior use required




mylife™ SafetyLancets Comfort
(30G/1.2 mm)


How to use the mylife™ SafetyLancet

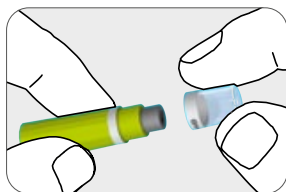


1 Simply twist and remove the protective cap from the mylife™ SafetyLancet.

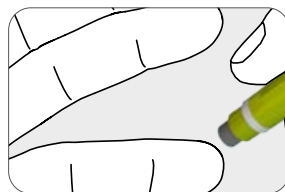

2 Place the mylife™ SafetyLancet on the finger.

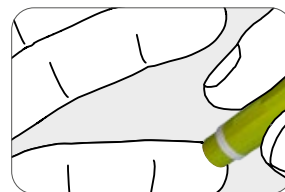

3 Push the raised platform firmly against the chosen site to activate the lancet mechanism.

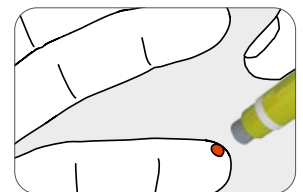

4 The needle retracts immediately after use. Thanks to a single-use release function the mylife™ SafetyLancet cannot be re-used inadvertently.



More freedom. More confidence. With mylife™.

YPSOMED
SELF CARE SOLUTIONS



mylife™ SafetyLancets

Sterile disposable safety lancets for a safe and comfortable routine medical blood sampling

Available in boxes of 100 and 200 pieces with convenient slider lid



mylife™ SafetyLancets (28G/1.5 mm)



mylife™ SafetyLancets Comfort (30G/1.2 mm)

Designed for blood glucose monitoring and other diagnostic tests in

- Diabetes clinics
- Hospitals
- Homecare

Maintains sterility for 5 years

- Sterilized by using irradiation
- Sterilising dose determined in accordance with ISO 11137

Gauge	Needle diameter	Penetration depth	Description	Quantity	Ref. No.
28G	0.36mm	1.5 ± 0.2 mm	mylife™ SafetyLancets	100 pcs	700009359
28G	0.36mm	1.5 ± 0.2 mm	mylife™ SafetyLancets	200 pcs	7100031
30G	0.30mm	1.2 ± 0.2 mm	mylife™ SafetyLancets Comfort	100 pcs	700009361
30G	0.30mm	1.2 ± 0.2 mm	mylife™ SafetyLancets Comfort	200 pcs	700001495

United Kingdom Ypsomed Ltd. // 1 Park Court // Riccall Road // Escrick // North Yorkshire // YO19 6ED // info@ypsomed.co.uk // www.mylife-diabetescare.co.uk // Customer Care: UK 0344 856 7820 or ROI 1800 911 522 (Calls to this number are free of charge from mobile phones and landlines, plus your phone company's access charge.)

India Ypsomed India Private Ltd. // 10/61/1-F, Kirti Nagar Industrial Area // New Delhi-110015 // info@ypsomed.co.in // www.mylife-diabetescare.co.in // Customer Care: 099999 75157