



Clickfine®  
AutoProtect™

# Enjoy prevention.



swissmade 

## mylife™ Clickfine® AutoProtect™ – the needle that only pricks when it's supposed to.

- Innovative locking mechanism for protection against needle stick injuries
- Supports EU 2010/32 §4, Art. 7 requirements<sup>1</sup>
- Cannula hidden by blue protection cap for more confidence
- Special silicone surface treatment for advanced injection comfort
- Compatible for all major injection pen brands<sup>2</sup>, available in 5 mm and 8 mm lengths

**AUTO  PROTECT**



More freedom. More confidence. With mylife™.

**YPSOMED**  
SELF-CARE SOLUTIONS



## We care about your safety

With its innovative and patented locking system, mylife™ Clickfine® AutoProtect™ minimises the risk of needle stick injuries, thus meeting the requirements of Council Directive 2010/32/EU §4, Art. 7<sup>1</sup>.

The mylife™ Clickfine® AutoProtect™ safety pen needle ensures following characteristics:

- Contains a fully integrated safety needle mechanism
- Automatically locks immediately after injection
- Red lock indicator clearly shows locked stage
- Visible cannula tip facilitates priming
- Lock permanently covers needle preventing repeated use



### 2010/32/EU §4, Art. 7 ✓

Supports the fulfilment of clause 4, article 7 of Council Directive 2010/32/EU of 10 May 2010 – Prevention from sharps injuries in the hospital and healthcare sector

Awarded 2009 by  
the Chicago Athenaeum:  
Museum of Architecture  
and Design



## Premium quality features

Experience reliable comfort



**Easy Glide**

Special silicone surface treatment enables constant gliding characteristics and smooth penetration.



**Easy Flux**

Top-quality medical cannula with thin-wall technology enables easy and optimal flux of medication because of its larger inner diameter.



## How it works

Reliable safety and simplicity



mylife™ Clickfine® AutoProtect™ with protective cap.



Before injection: cannula protected by protective sleeve.




During injection: protective sleeve releases needle.




After injection: automatic locking and red locking indicator.




Clickfine®  
AutoProtect™

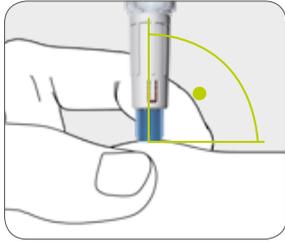


Insert the injection pen at a 90° angle to the desired injection site, keeping it vertical.

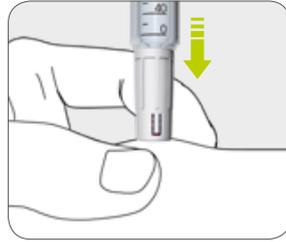

Exert a gentle pressure on the injection pen until the blue protective sleeve is no longer visible.

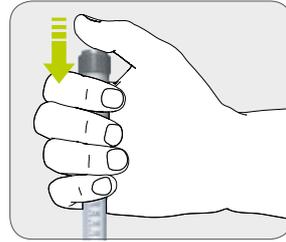

Now press the dose button and slowly administer the injection. Allow the pen needle to remain in the fatty tissue (count to 10 slowly).

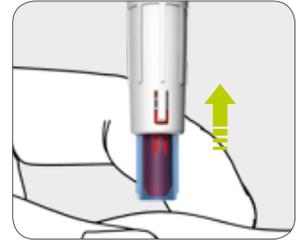

Remove the injection pen. The blue protective sleeve locks automatically and the red locking indicator becomes visible.



Consult instructions for use



## Choose the preferred needle length for your patients

Colour coding for simple identification of different needle lengths

The mylife™ Clickfine® AutoProtect™ pen needle is available in two different lengths: 5 mm and 8 mm.

The 5 mm needle may be used without a skin fold technique minimising the risk of an accidental needle stick to caregivers during injection.<sup>3</sup>



5 mm, 31 G / 0.25 mm



8 mm, 29 G / 0.33 mm



## Universal click®

mylife™ Clickfine® AutoProtect™ fits all major injection pen brands

mylife™ Clickfine® AutoProtect™ pen needles are tested according to ISO 11608-2:2012. They are fully functional and provide better safety when used with the injection pens listed on the mylife™ Clickfine® AutoProtect™ pen needle box. If in doubt please contact the injection pen manufacturer. Follow the instructions for use of your injection pen before using a mylife™ Clickfine® AutoProtect™ pen needle.



<sup>1</sup> Council Directive 2010/32/EU – Prevention from sharp injuries in the hospital and healthcare sector.

<sup>2</sup> See the list of compatible pens on the mylife™ Clickfine® AutoProtect™ pen needle box or website [www.mylife-diabetescare.com](http://www.mylife-diabetescare.com).

<sup>3</sup> K. Strauss and WISE Consensus Group: WISE recommendations to ensure the safety of injections in diabetes. Diabetes & Metabolism. 2012, 38, S2–S8, Supplement 1.



**Diabetescare**

More **freedom.**  
More **confidence.**  
With **mylife™.**



Infusion systems



Blood glucose  
monitoring systems



Therapy  
management



Pen needles

With its mylife™ Diabetescare brand, Ypsomed offers a comprehensive portfolio of products and services for people with diabetes. This allows users easy, discreet and reliable self-treatment. With mylife™ Diabetescare, self-treatment becomes simpler and easier.

**United Kingdom** Ypsomed Ltd. // 1 Park Court // Riccall Road // Escrick // North Yorkshire // YO19 6ED //  
info@ypsomed.co.uk // www.mylife-diabetescare.co.uk // Customer Care: 0344 856 7820  
(Calls to this number are free of charge from mobile phones and landlines,  
plus your phone company's access charge.)

**India** Ypsomed India Private Ltd. // 10/61/1-F, Kirti Nagar Industrial Area // New Delhi-110015 //  
info@ypsomed.co.in // www.mylife-diabetescare.co.in // Customer Care: 099999 75157